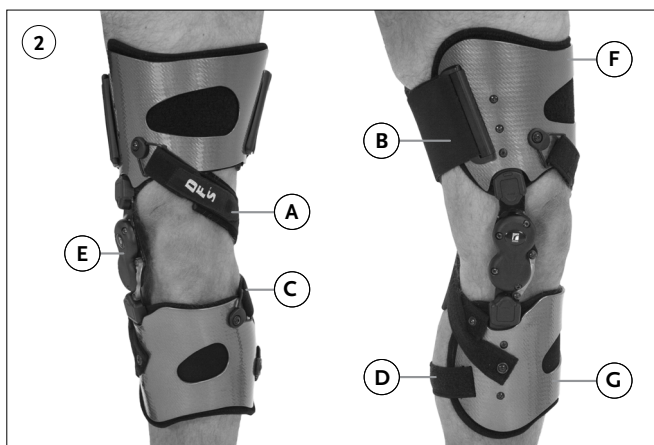
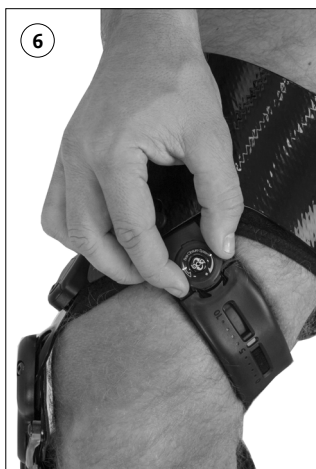
Instructions for Use

UNLOADER[®] CUSTOM

		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	9
FR	Notice d'utilisation	13
ES	Instrucciones para el uso	17
NL	Gebruiksaanwijzing	21

ENGLISH

DISCLAIMER

The Unloader® Custom is a truly custom brace fit to the patient's needs. Each patient has a unique product specifically fit to the patient's anatomy, activity level and desired configuration. Therefore images within the Instructions for Use may appear different than the product the patient may have received.

LIST OF PARTS

Unloader Custom with SmartDosing® (Figure 1)

- A. Dynamic Force Strap (DFS)
- B. Thigh strap
- C. Mid strap
- D. Calf strap
- E. Hinge
- F. Thigh shell
- G. Calf shell
- H. Quick Fit Buckle
- I. Alligator clip
- J. SmartDosing® dial

LIST OF PARTS

Unloader Custom with Standard DFS (Figure 2)

- A. Dynamic Force Strap (DFS)
- B. Thigh strap
- C. Upper calf strap
- D. Lower calf strap
- E. Hinge
- F. Thigh shell
- G. Calf shell

INDICATIONS FOR USE

- Unicompartmental Osteoarthritis
- May also be beneficial for unicompartmental knee conditions that require pain reduction in an affected compartment such as:
 - Articular cartilage defect repair
 - Avascular necrosis
 - Tibia plateau fracture

CAUTION

- **This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions**
- For patients with peripheral vascular disease, neuropathy, and sensitive skin, close physician supervision is recommended. For patients with sensitive skin, it is recommended to use the Unloader® Custom as tolerated. If redness appears decrease the usage of the brace. Increase usage slowly as the skin tolerates the brace.
- The Unloader® Custom is field adaptable to accommodate patient comfort issues. Modifications to flexion or extension stops and shell contour should only be made by the health care professional. The healthcare professional is responsible for advising patient on use of the Dynamic Force Control System.

BRACE APPLICATION (FOR SMARTDOSING® OPTION)

1. Place the brace on patient's leg. Align the center of hinge at mid patella and midline of A/P
2. Wrap the upper calf strap around the back of your leg above the calf muscle, thread it through the D-ring and secure **(Figure 3)**.
3. Hold the thigh shell and bring the strap around leg. Thread the thigh strap through the slotted thigh strap attachment and secure **(Figure 4)**.
4. Place button of Quick Fit Buckle into appropriate keyhole **(Figure 5)** and close carefully using the palm of the hand to snap the buckle closed.
5. Turn SmartDosing® dial clockwise to tighten Dynamic force Control System strap to prescribed position **(Figure 6)**.
Practitioner will need to determine optimum unloading based on patient's pain relief feedback. The strap tension should be set to provide optimal pain relief once the dosing dial is set to the setting of 5.

BRACE APPLICATION (FOR STANDARD DFS OPTION)

1. Place the brace on patient's leg. Align the center of hinge at mid patella and midline of A/P.
2. Wrap the upper calf strap around the back of your leg above the calf muscle, thread it through the D-ring and secure **(Figure 3)**.
3. Hold the thigh shell and bring the strap around leg. Thread the thigh strap through the slotted thigh strap attachment and secure **(Figure 4)**.
4. Thread the lower calf strap through the D-ring and secure **(Figure 7)**.
5. Bend your braced knee to an approximate angle of 80 degrees. While holding the thigh shell, to prevent the brace from moving, thread the DFS strap through D-ring in front of thigh shell and tighten **(Figure 8)**.
The DFS should be worn snug for maximum pain relief.

INTERNAL UNI-CENTRIC ADJ

The braces are shipped with a Range of Motion (ROM) Stop in Place. For this setting the ROM Stop is positioned with the long end down (relative to the hole in the stop). If the Range of Motion needs to be adjusted, first remove the Upper Hinge Cover screw using the supplied Allen Key. Remove the Upper Hinge Cover to access the ROM mechanism. Set the ROM Stop and ROM Screw to the desired position as indicated below.

0 degrees

Remove ROM Stop. Orient ROM Stop with long end down. Insert ROM Stop Screw in distal stop hole.

5 degrees

Remove ROM Stop. Orient ROM Stop with long end up. Insert ROM Stop Screw in distal stop hole.

10 degrees

Remove ROM Stop. Insert ROM Stop Screw in the proximal stop hole.

20 degrees

Remove ROM Stop. Orient ROM Stop with long end down. Insert ROM Stop Screw in the proximal stop hole.

-10 degrees hyperextension

Remove ROM Stop. Insert ROM Stop Screw in the distal stop hole.

POLI-AXIAL ADJ™ HINGE

Things to know before adjusting the brace:

To treat either the medial or lateral compartment osteoarthritis, the brace has a user-friendly ADJ™ hinge mechanism, which can be adjusted to optimize the unloading effect in either the medial or lateral compartment of the knee.

The degree of adjustment required depends on the patient's symptoms, weight, musculature, ease of joint manipulation, and severity of joint degeneration.

For example: More hinge correction may be needed for patients with an increase in symptoms or an increase in Body Mass Index (BMI).

A TRAINED DEALER OR CLINICIAN SHOULD MAKE ALL HINGE ADJUSTMENTS. ÖSSUR DOES NOT RECOMMEND THAT PATIENTS MAKE ADJUSTMENTS.

Hinge adjustment

1. Remove the ADJ™ cover; this will expose the Adjusting Mechanism, Adjusting Screw (A), and Locking Screw (B).
2. Adjust one ADJ™ Mechanism at a time.
3. Use a Robertson Screwdriver to back the Locking Screw out approximately two full turns counterclockwise (do not remove completely).
4. Use a Robertson Screwdriver to turn Adjusting Screw clockwise to increase hinge angle or counter-clockwise to decrease hinge angle.
**Three flat sides (1/2 turn) of the Adjusting Screw represents approximately one degree adjustment, therefore, one complete turn of the screw equals approximately two degrees of adjustment.*
Note: over-adjustment may result in brace rotation and/or excessive pressure. Discrepancy between proximal and distal correction may result in altered fit of brace.
5. Apply a thread lock compound to the locking screw.
6. Retighten the locking screw snug to the hinge. Do not overtighten as this will cause premature hinge failure.
7. Repeat the adjustment procedure for the other ADJ™ Mechanism.
8. When complete, replace the ADJ™ covers.
9. Correctly re-apply the brace to the patient's leg and assess the fit. Ensure the DFS™ strap is applied with sufficient tension to augment the unloading effects of the ADJ™ hinge.

BRACE MAINTENANCE

Unloader® Custom is designed for minimum maintenance and care. Follow these easy maintenance steps to keep brace in good condition.

CLEANING

- Sensil® liners may be wiped down with disinfectant liquid to prevent build-up of harmful bacteria and with antibacterial detergent once a week.
- For more thorough cleaning: liners, straps and strap pads may be removed.
- Hand-wash using mild non-detergent soap
- Rinse thoroughly and air dry
- **Do not machine wash**

- **Do not place in dryer**

HINGE

- For safety reasons, hinge should only be disassembled by a healthcare professional.
- Check regularly for foreign materials in hinge (i. e. dirt or grass).
- Clean dirt from hinge using tap water and/or compressed air.

Note: If brace is used in salt water or chlorinated water, rinse well with fresh tap water and air dry brace. A regular service routine will ensure hinge, liners, straps and pads are in optimal working condition.

BRACE ACCESSORIES

Össur offers accessories that meet specific needs of the Unloader® Custom. These accessories can be purchased through the brace provider or Össur Customer Service Department.

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